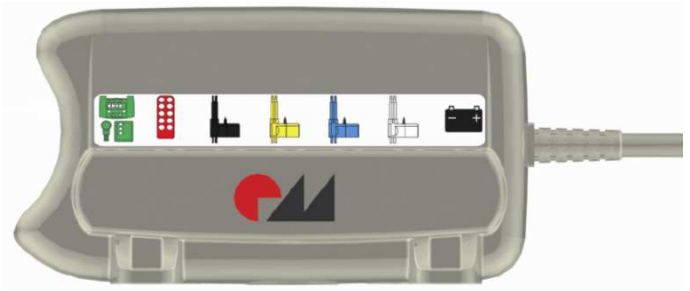
MultiControl Care mono

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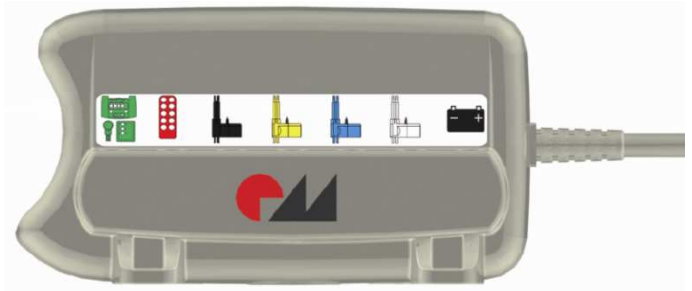
10-Tasten-IPROXX

MultiControl Care mono

Supplement for the assembly instructions for control MCL II / IPROXX hand switch

Connecting the drives

Up to four Mono drives (connection A) can be connected to the MultiControl Care mono and traversed. The drives do not have to be initialised.

Unlocking the IPROXX hand switch using a magnetic key

Before operation, the 8 and 10 button IPROXX hand switches must be unlocked using the magnetic key - for this purpose, pull the magnetic key over the lock symbol on the bottom part of the hand switch.

On the 10 button IPROXX hand switch, the lock LED is switched to green after a 1-fold unlocking. The lock LED is switched to orange after 2-fold unlocking. The hand switch is locked after approx. 1 minute or automatically after pulling the magnetic switch over for the 3rd time - the lock LED goes out.

On the 8 button IPROXX hand switch, the lock LED is switched to orange after a 1-fold unlocking. The hand switch is locked after pulling the magnetic key over for the 2nd time - the lock LED goes out.

Magnetic key

Operation

The drives on the slots marked blue and white can be traversed with the following hand switches or foot switches. In doing so, these two drives are traversed parallelly (not synchronous). These drives can also be traversed with the bottom buttons of the 10 button IPROXX hand switch antiparallely (anti-Trendelenburg after 1-fold, Trendelenburg after 2-fold unlocking). When assembling, observe the respective floating bearing.

With **anti / Trendelenburg – adjustment**, the customer is responsible to observe the permissible inclination in the respective application.

2-Button-Classic

Foot Switch

8-Button-IPROXX

10-Button-IPROXX

The drives on the slots marked black and white can be controlled separately or parallelly using the 6-Button-Classic hand switch or IPROXX hand switch. The respective hand switches are represented in the following images.

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